F. BOSCH

---

MEDIZINTECHNIK  
GERMANY

**Instructions For Use**

## ***Therapin Soft Active***<sup>®</sup>

**Product:** Vaginal Speculum with side screw,  
Small/Medium/Large

**Article No.:** FBHEAL-1012, 1013, 1014

## Pictogram Annex

<sup>\*1</sup>  
Instructions for use

<sup>\*2</sup>  
XXXX = Identification number of notified body responsible for the EU type examination and supervising ongoing conformity

<sup>\*3</sup>  
For single-use only, must not be reused

<sup>\*4</sup>  
Lot Number

<sup>\*5</sup>  
Date of Manufacture

<sup>\*6</sup>  
Manufacturer

<sup>\*7</sup>  
Expiration Date

<sup>\*8</sup>  
Sterilized using ethylene oxide

<sup>\*9</sup>  
Avoid humidity

<sup>\*10</sup>  
Keep away from direct sunlight

<sup>\*11</sup>  
Disposal according to legislation

<sup>\*12</sup>  
Medical Device

<sup>\*13</sup>  
Do not use if package is damaged

## English

Lubricate the device before using it. Introduce slowly and gently the speculum into the vagina with the two plastic clamps closed. Press delicately the two handles until they get close to each other. Screw the pin until it secures without pressing with the hand. Unscrew so that the two plastic clumps can close and remove it from the vagina gently.

**CAUTION:** The device should be used by professionals. Use it immediately after opening the package.

**INTENDED USE:** The device should be used during gynecological examinations, for the vaginal inspections, for the dilatation of the vaginal canal and for the exposition of the uterine neck.

\*1) Instructions for use, \*2) XXXX = Identification number of notified Body responsible for the EU type examination and supervising ongoing conformity, \*3) For single-use only, must not be reused, \*4) Lot Number, \*5) Date of Manufacture , \*6) Manufacturer, \*7) Expiration Date, \*8) Sterilized using ethylene oxide, \*9) Avoid humidity, \*10) Keep away from direct sunlight, \*11) Disposal according to legislation, \*12) Medical Device, \*13) Do not use if package is damaged

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**Disclaimer:**

The product information contained in this document, including the product images and additional product material, was collected from various sources. All product information and specifications have not been independently verified by F. Bosch International Ltd. F. Bosch International Ltd is not responsible for errors or omissions in the product information. The properties of a product may change or be inaccurate on the posting or printing of the product information in the document, either in printed or online version. Buyers and users should be cautious when using or purchasing any of the products mentioned in the printed or online versions of the documents by closely examining the product packaging and the labeling prior to use. Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in placing orders and F. Bosch International Ltd disclaims all responsibility for its completeness and accuracy. © 2023 F. Bosch International Ltd.