F. BOSCH

---

MEDIZINTECHNIK  
GERMANY

**Instructions For Use**

***Therapin Soft Active***®

**Product: Sterilization roll**

**Article No.: FBHEAL-1044, 1045, 1046, 1047, 1048, 1049**

## Pictogram Annex

\*1  
Instructions for use

\*2  
Indicates that the product meets EU safety, health and environmental protection requirements

\*3  
For single-use only, must not be reused

\*4  
Lot Number

\*5  
Date of Manufacture

\*6  
Manufacturer

\*7  
Expiration Date

\*8  
Non-sterile

\*9  
Avoid humidity

\*10  
Keep away from direct sunlight

\*11  
Disposal according to legislation

\*12  
Medical Device

\*13  
Hazardous product, can cause certain irritations

\*14  
Do not use if package is damaged

\*15  
The producer has made a financial contribution to a packaging recovery system

## English

Cut the sterilization roll at the appropriate length so that your instrument can fit inside without damaging it. Before sealing, remove as much air as possible from the roll to avoid rupturing. Place the roll in the sterilization device of your choice, following the user's manual. When completed, remove the roll and ensure that the instrument is sterilized by checking the color indicator. The blue indicator will turn dark grey when sterilized by steam. The pink indicator will turn yellow when sterilized by EO.

\*1) Instructions for use, 2\*) Indicates that the product meets EU safety, health and environmental protection requirements \*3) For single-use only, must not be reused, \*4) Lot Number, \*5) Date of Manufacture , \*6) Manufacturer, \*7) Expiration Date, \*8) Non-sterile, \*9) Avoid humidity, \*10) Keep away from direct sunlight, \*11) Disposal according to legislation, \*12) Medical Device, \*13) Hazardous product, can cause certain irritations, \*14) Do not use if package is damaged, \*15) The producer has made a financial contribution to a packaging recovery system

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### Disclaimer:

The product information contained in this document, including the product images and additional product material, was collected from various sources. All product information and specifications have not been independently verified by F. Bosch International Ltd. F. Bosch International Ltd is not responsible for errors or omissions in the product information. The properties of a product may change or be inaccurate on the posting or printing of the product information in the document, either in printed or online version. Buyers and users should be cautious when using or purchasing any of the products mentioned in the printed or online versions of the documents by closely examining the product packaging and the labeling prior to use. Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in placing orders and F. Bosch International Ltd disclaims all responsibility for its completeness and accuracy. © 2023 F. Bosch International Ltd.