F. BOSCH

---

MEDIZINTECHNIK  
GERMANY

**Instructions For Use**

***Therapin soft Active***®

**Product: Plaster of Paris Bandage**

**Article No.: FBHEAL-1050, 1051, 1052, 1053**

## Pictogram Annex

<sup>\*1</sup>  
Instructions for  
use

<sup>\*2</sup>  
Indicates that the product  
meets EU safety, health and  
environmental protection  
requirements

<sup>\*3</sup>  
For single-use only,  
must not be reused

<sup>\*4</sup>  
Lot Number

<sup>\*5</sup>  
Date of  
Manufacture

<sup>\*6</sup>  
Manufacturer

<sup>\*7</sup>  
Expiration  
Date

<sup>\*8</sup>  
Non-sterile

<sup>\*9</sup>  
Avoid humidity

<sup>\*10</sup>  
Keep away from  
direct sunlight

<sup>\*11</sup>  
Disposal  
according to  
legislation

<sup>\*12</sup>  
Medical Device

## English

Before applying the P.O.P bandage, cover the skin area with stocking net and/or orthopedic padding. Soak only one roll at a time and immerse the bandage at 45° slant for 5-15 seconds until bubbling ceases. Remove the bandage from the water and squeeze out the excess water, pressing the bandage softly from both edges towards the middle. If a second bandage is required, lightly squeeze the first one to leave enough water to soak on the second bandage so that both bandages are properly merged and formed appropriately. Shape the bandage by hand continuously during wrapping. Wrapping process must be completed before setting time. After the shaping & wrapping process, the bandage will take 2-3 minutes to harden so that the bandage can be properly used. During this time, no movement of the covered area is allowed.

\*1) Instructions for use, \*2) Indicates that the product meets EU safety, health and environmental protection requirements \*3) For single-use only, must not be reused, \*4) Lot Number, \*5) Date of Manufacture, \*6) Manufacturer, \*7) Expiration Date, \*8) Non-sterile, \*9) Avoid humidity, \*10) Keep away from direct sunlight, \*11) Disposal according to legislation, \*12) Medical Device

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### Disclaimer:

The product information contained in this document, including the product images and additional product material, was collected from various sources. All product information and specifications have not been independently verified by F. Bosch International Ltd. F. Bosch International Ltd is not responsible for errors or omissions in the product information. The properties of a product may change or be inaccurate on the posting or printing of the product information in the document, either in printed or online version. Buyers and users should be cautious when using or purchasing any of the products mentioned in the printed or online versions of the documents by closely examining the product packaging and the labeling prior to use. Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in placing orders and F. Bosch International Ltd disclaims all responsibility for its completeness and accuracy. © 2023 F. Bosch International Ltd.