MEDIZINTECHNIK
GERMANY

Instructions For Use

Therapin soft Active[®]

Product: Plastic cotton applicator Stuart, sterile

Article No.: FBHEAL-1055

Pictogram Annex

*1
Instructions for use

*2
XXXX = Identification number of notified body responsible for the EU type examination and supervising ongoing conformity

*3
For single-use only, must not be reused

*4
Lot Number

*5
Date of Manufacture

*6
Manufacturer

*7
Expiration Date

*8
Sterilized using irradiation

*9
Avoid humidity

*10
Keep away from direct sunlight

*11
Disposal according to legislation


*12
Medical Device

*13
Fragile

*14
Do not use if package is damaged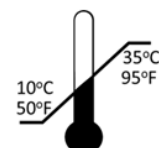

*15
Storage temperature limitations

English

Open the sterile pouch at the point marked "Peel Here". Remove the cap from the transport tube. Use the applicator swab to collect specimen. During specimen collection, the applicator tip should only touch the area where the suspected infection is to minimize potential contamination. Place the applicator swab in the transport tube and replace the cap firmly to seal completely. Record patient's name and information on the tube label.

*1) Instructions for use, *2) XXXX = Identification number of notified Body responsible for the EU type examination and supervising ongoing conformity, *3) For single-use only, must not be reused, *4) Lot Number, *5) Date of Manufacture, *6) Manufacturer, *7) Expiration Date, *8) Sterilized using irradiation, *9) Avoid humidity, *10) Keep away from direct sunlight, *11) Disposal according to legislation, *12) Medical Device, *13) Fragile, *14) Do not use if package is damaged, *15) Storage temperature limitations

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Disclaimer:

The product information contained in this document, including the product images and additional product material, was collected from various sources. All product information and specifications have not been independently verified by F. Bosch International Ltd. F. Bosch International Ltd is not responsible for errors or omissions in the product information. The properties of a product may change or be inaccurate on the posting or printing of the product information in the document, either in printed or online version. Buyers and users should be cautious when using or purchasing any of the products mentioned in the printed or online versions of the documents by closely examining the product packaging and the labeling prior to use. Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in placing orders and F. Bosch International Ltd disclaims all responsibility for its completeness and accuracy. © 2023 F. Bosch International Ltd.