F. BOSCH

---

MEDIZINTECHNIK  
GERMANY


**Instructions For Use**

***Therapin soft Active***®

**Product: Intravenous Infusion Set**

**Article No.: FBHEAL-1067**

## Pictogram Annex

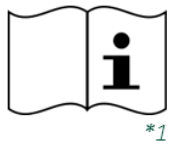<sup>\*1</sup>  
Instructions for  
use

<sup>\*2</sup>  
XXXX = Identification  
number of notified  
body responsible for  
the EU-type  
examination and  
supervising ongoing  
conformity

<sup>\*3</sup>  
For single-use  
only, must not  
be reused

<sup>\*4</sup>  
Lot Number

<sup>\*5</sup>  
Date of  
Manufacture

<sup>\*6</sup>  
Manufacturer

<sup>\*7</sup>  
Expiration  
Date

<sup>\*8</sup>  
Sterilized using  
ethylene oxide

<sup>\*9</sup>  
Do not use if  
package is  
damaged

<sup>\*10</sup>  
Hazardous product,  
can cause certain  
irritations

<sup>\*11</sup>  
Medical Device

## English

1. Open the individual package and take out the intravenous infusion set
2. Firmly close roller clamp
3. Remove drip chamber's protective cap
4. Insert plastic spike into the solution container
5. Half fill the drip chamber with solution by gently squeezing and releasing the drip chamber
6. Open roller clamp and expel air bubbles
7. Close roller clamp again
8. Place the connector into the vein cannula
9. Use the roller clamp to adjust flowing rate

\*1) Instructions for use, \*2) XXXX = Identification number of notified Body responsible for the EU type examination and supervising ongoing conformity, \*3) For single-use only, must not be reused, \*4) Lot Number, \*5) Date of Manufacture, \*6) Manufacturer, \*7) Expiration Date, \*8) Sterilized using ethylene oxide, \*9) Do not use if package is damaged, \*10) Hazardous product, can cause certain irritations, \*11) Medical Device

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### Disclaimer:

The product information contained in this document, including the product images and additional product material, was collected from various sources. All product information and specifications have not been independently verified by F. Bosch International Ltd. F. Bosch International Ltd is not responsible for errors or omissions in the product information. The properties of a product may change or be inaccurate on the posting or printing of the product information in the document, either in printed or online version. Buyers and users should be cautious when using or purchasing any of the products mentioned in the printed or online versions of the documents by closely examining the product packaging and the labeling prior to use. Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in placing orders and F. Bosch International Ltd disclaims all responsibility for its completeness and accuracy. © 2023 F. Bosch International Ltd.